

Dolphin Therapy is a reactive mattress system that creates a simulated fluid environment, enabling full immersion and envelopment, significantly reducing pressure, shear and tissue deformation.

Key Benefits:

- Tissue symmetry is maintained
- Vascular occlusion is minimised, maintaining near normal blood flow
- Tissue ischemia is prevented, even when the patient is nursed directly on the affected area



Call **08 45 371 1717** to order or visit www.medstrom.com/dolphin-therapy for more information

Bariatric Equipment Overview

There is a Trust wide fund for hiring bariatric equipment. Please use cost code 0036065 when ordering equipment. **Ward budget codes are no longer required.**

If equipment is not hired the Moving & Handling Team would expect to see a written risk assessment highlighting how the patient is being cared for safely with the necessary equipment.

All equipment required for hire must be ordered from Benmor Medical out of the central budget code.

Tel: 0333 800 9000

Web: www.benmormedical.co.uk

BENMOR MEDICAL will hire out beds, commodes, chairs and other equipment. Delivery is usually within 4 hours so the earlier you can get the information to Benmor, the better.

Delivery is 24/7 & 365 days of the year.

Out of hours

When you may not have access to i-proc, please telephone BENMOR MEDICAL. An order number

FootSafe is for patients that are assessed as 'at risk of pressure ulcers'

FootSafe boots are supplied as a pair and marked left and right

The inflated boot will lift the heel and off load pressure

Be careful not to over-inflate as this is likely to result in patient discomfort

Do not over-tighten the straps. You should be able to slide a finger between the straps and the patients legs

Do not allow the patients to walk in the boots

Inspect the skin regularly, re-evaluate suitability frequently and step up if required

May be decontaminated and re-used



| Description | NHS SC Code |
|--|-------------|
| FootSafe Prevention Boot – Uncovered – Inflatable NO Pump – Childs | GRA5219 |
| FootSafe Prevention Boot – Uncovered – Inflatable NO Pump – Youth | GRA5224 |
| FootSafe Prevention Boot – Uncovered – Inflatable NO Pump – Small Adult | GRA3770 |
| FootSafe Prevention Boot – Uncovered – Inflatable NO Pump – Standard Adult | GRA3772 |
| FootSafe Prevention Boot – Uncovered – Inflatable NO Pump – X Large Adult | GRA3774 |
| FootSafe Hand Pump Only (Single) | GRA5208 |

HeelPro is for patients with existing pressure ulcers of Category 2 Plus

HeelPro boots are supplied individually and not as a pair. Each boot is interchangeable can be used on both the left and right foot

Do not overtighten Lycra panels, see multiple ways of adjustment

For lightly contaminated boots disinfectant wipe can be used to clean

Discard if heavily soiled or boot integrity compromised

Do not allow the patients to walk in the boots

Single patient use only

| Description | NHS SC Code |
|---------------------------------------|-------------|
| HeelPro Heel Protection Boot – Single | GRD1502 |
| HeelPro Heel Protection Boot – Box 10 | GRA1664 |



Bed



Air Cushion

Mattress Options

Dynamic



Turning

