

Servo – U Turnaround Guide

- Switch off ventilator – switch is found on the back, inside the “door” – Metal switch needs to be pulled down slightly and flicked to the left.
- Remove all old tubing and throw away.
- Do not throw away cream coloured nebuliser cable, or dark grey Test tubing.
- Ensure ventilator is cleaned properly, including Expiratory cassette.
 - To remove cassette push rectangular button, lift and slide out towards you.
 - The outer case needs cleaning, including white rubber seal underneath and allow to dry.
 - Slide expiratory cassette back into ventilator and push down on top (not on expiratory connection) until it clicks in place.
- Connect dark grey test tube to inspiratory and expiratory connections, plug in to mains and switch on ventilator.
- Once on the vent will ask you if you want to perform device check, press “Yes” on screen.
- Follow on screen instructions.
- Last part of check will need a yellow bacterial filter on expiratory connection, normal breathing circuit connected to Exp and Insp connection, HME filter on “Y-Piece” and short catheter mount on HME.
- Vent will ask you to connect all of these to the vent and seal catheter mount to perform “patient circuit test” – follow on screen instructions.
- If you switch off the ventilator after calibrations and self-test, this will then be “erased” and will need to be performed again once switched on.
- These tests only need to be performed if the ventilator is going to be left on standby.