




External pacing
(See overleaf for algorithm)

1. Attach 3 lead ECG and defibrillation pads
2. Turn dial to **Pacer** using the control dial  highlight and select **Pacer Rate**.
3. Turn control dial to select desired rate  Press control dial to confirm rate.
4. Using control dial select **Pacer Output** 
5. Turn Control dial to increase current until **electrical capture**.
6. **Palpate central pulse** to confirm **mechanical capture**. Press control dial to confirm current.

If necessary, increase Pacer Output until mechanical capture.
Increase **Pacer Output** by 10% and continuously monitor central pulse.
7. Consider **sedation** and/or **analgesia** if patient uncomfortable.



Adult bradycardia

