

## ETCO<sub>2</sub> monitoring

**WARNING – Patient must be able to tolerate removal of 50mL/min from breathing circuit without inducing hypercapnia**

You will need –

- CapnoFlex module
- CapnoStat module
- Airway adapter



### Set-up

- Connect the CapnoFlex module to the CapnoStat module as shown



- Slide into any channel of the monitor rack
  - **WARMING UP** is displayed
- When finished warming up, connect airway adapter to the underside of the CapnoFlex module



- On monitor, select **CO<sub>2</sub>** menu > **CAL SENSOR TO ZERO CELL** > **READY**
  - **CALIBRATING** is displayed
- Wait for message to disappear & connect adapter into patient circuit between flow sensor & ETT
- Keep tubing pointing up & routed above baby to reduce rain-out