


Fabian ventilator

Set-up for tubing

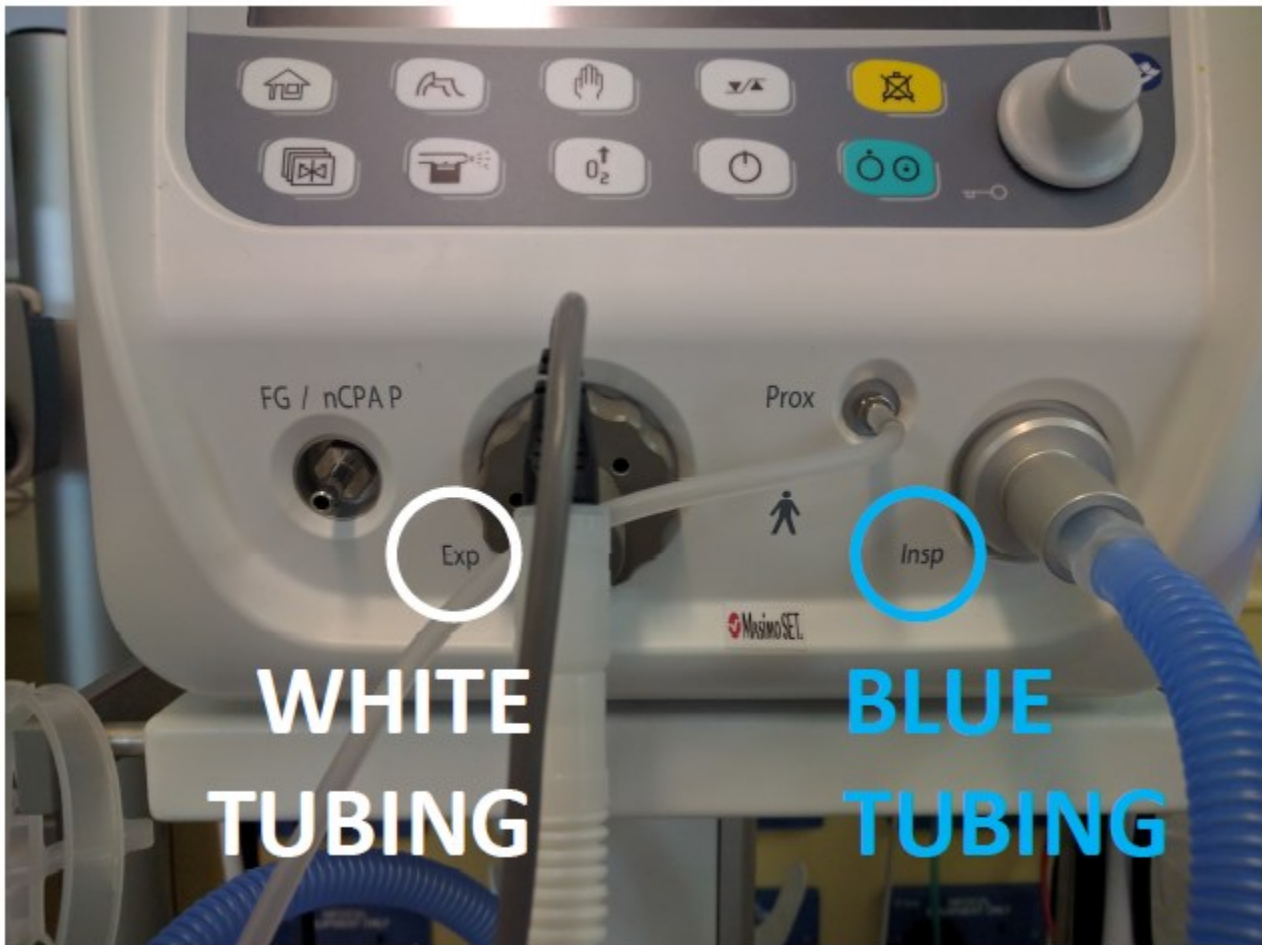


- Blue inspiratory tubing should be placed on the right of the ventilator for invasive ventilation
- White expiratory tubing should be placed on the left of the ventilator for invasive ventilation

Ventilation / high frequency oscillation (HFO)

| Mode | Description |
|------------------------|--|
| SIPPV | Assist/control ventilation |
| CPAP | CPAP via ventilator circuit or Argyle prongs |
| NCPAP | CPAP via Infant Flow circuit |
| DUOPAP | SiPAP via Infant Flow circuit |
| O ₂ Therapy | Use for Neopuff (only non-NO patients) HHFNC / Bubble CPAP |

- You need
 - New flowsensor
 - F&P RT265 breathing circuit
 - Clean expiratory membrane & holder (in SSD pack)
 - Keep the tag to return to SSD with the membrane & holder**
- Assemble expiratory membrane & holder as shown → → →
 - Place the membrane in the holder so the word **TOP** is visible in the centre
 - Place the holder in position and turn it clockwise to secure
- Assemble breathing circuit
- Flowsensor
 - Attach to flowsensor cable
 - Attach to breathing circuit
 - When the Fabian is powered up, calibrate it
 - It will need to be in a ventilation mode which uses the flowsensor**

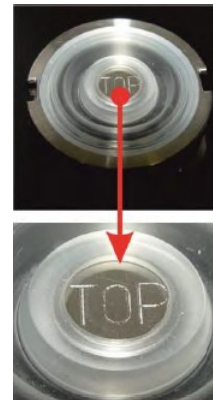


Non-invasive ventilation (NIV), e.g. HHFNC / Bubble CPAP / Infant Flow CPAP

- You need
 - The correct breathing circuit
 - Fabian NIV adapter
 - Expiratory membrane & holder is not required for NIV**
- Place NIV adapter between the **FG / nCPAP** prong and the humidifier chamber
 - You do not use the white pressure manifold, but keep it for when the patient steps down into N2/3**
- Assemble breathing circuit

Cleaning

- The items shown below are **not** disposable
- Any **other** part of the breathing circuit **is** disposable, **excluding the NO injector module**, if used

Expiratory membrane & holder **(to SSD)**

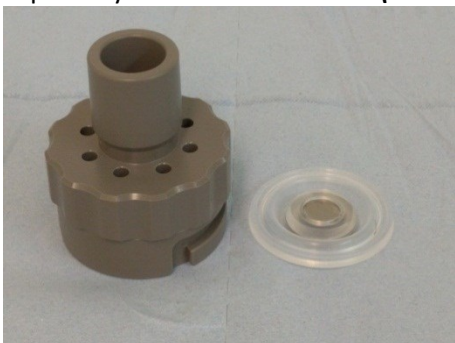

Flowsensor cable **(stays attached to vent)**



Expiratory membrane & holder

- Replace the expiratory membrane & holder between patients
- To remove the holder, twist anti-clockwise
- Place both parts in SSD bag with their tag
 - Put the bag in the red SSD box in the cleaning room

Flowsensor wire

- Should not be unscrewed from the back of the vent, does not go to SSD**
- Clean with green Clinell wipe, following unit protocols
- Coil it loosely around the left hand rail